

## CESIUM DR UNIVERSAL TECHNIQUE CHART

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Hip	AP	85	4 to 5	85	8	85	12
Hip	X-Table Lat (Grid)	90	14	90	20	90	36
Humerus	AP (Non Grid)	66	1.6	66	2	66	2.5
Knee	AP (Bucky)	77	3.2	77	4	77	6.4
Knee	Lat (Bucky)	77	2.5	77	3.2	77	5
Knee	Sunrise	70	1.6	70	2	70	2.5
Knee	Non-Bucky	70	2.5	70	3	70	3.6
L-Spine	AP	90	4	90	8	90	14
L-Spine	X-Table Lat (Grid)	95	16	95	28	95	40
Mandible	Obl (40")	81	8	81	10	81	12
Pelvis	AP (Grid)	85	8	85	12	85	16
Ribs	Upper AP (72")	81	6	81	12	81	18
Ribs	Upper Obl (72")	81	8	81	16	81	25
Ribs	Lower AP (45")	85	6.0	85	12	8	18
Shoulder	AP (Bucky)	77	4	77	7	77	12
Shoulder	Mercedes	77	8	77	15	77	25
Shoulder	X-T Axillary (N-Grid)	70	3	70	3.5	70	4
Sinus	Caldwell	85	4	85	6	85	8
Sinus	Waters	85	5	85	7	85	9
Sinus	Lateral	85	2	85	3	85	4
Skull	AP	85	4	85	6	85	8
Skull	Lat (Grid)	85	2.5	85	3.2	85	4
Tib-Fib	AP	77	2	77	2.5	77	3.2
Tib-Fib	Lat	77	1.6	77	2	77	2.5
Toe	All Views	60	0.63	60	0.8	60	1
T-Spine	AP	85	5	85	8	85	11 to 12
T-Spine	Lat	90	10	90	18	90	28
Wrist	PA	63	0.8	63	1	63	1.25
Wrist	Obl	63	1	63	1.25	63	1.5
Wrist	Lat	67	1.2	67	1.5	67	1.8
Zygomatic Arch	SMV view	70	5 to 6	70	8	70	10



Digital Radiography Solutions "Specializing in Educational Conferences and Seminars"

**Disclaimer** - Digital Radiography Solutions (DRS) cannot be held responsible for any images taken or passed in your facility. Your radiologist always has the final say if an image is diagnostic.