

KONICA CR TECHNIQUE CHART Least mAs

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Hip	AP (non grid)	70	3.2	70	4	70	5
Hip	X-Table Lat (Grid)	90	16 - 25	90	30 - 40	90	50 - 60
Humerus	AP (non grid)	65	1.25	65	2	65	2.5
Knee	AP (Bucky)	75	2.7	75	3.6	75	4
Knee	Obl (Bucky)	75	2.4	75	3.2	75	3.6
Knee	Lat (Bucky)	75	2.4	75	3.2	75	3.6
Knee	Sunrise (non grid)	65	3	65	3.8	65	4.8
Knee	Non-Bucky	65	2.7	65	3	65	3.3
L-Spine	AP	80	7 to 11	80	14 to 18	80	22 - 26
L-Spine	X-Table Lat (Grid)	90	55 - 65	90	85 - 100	90	160 - 200
Mandible	Obl (non grid - 40")	65	4.5	65	6	65	7.5
Pelvis	AP (Grid)	80	7.3	80	14.5	80	23
Ribs	Upper (72")	75	8 to 12	75	14 to 20	75	25 - 32
Ribs	Lower (40")	80	5.0 - 8	80	10 - 12.5	80	16 - 20
Ribs	Obl Upper (72")	75	7 to 9	75	14 - 22	75	23 - 28
Shoulder	AP (non grid)	65	2.4	65	3	65	3.2
Shoulder	Mercedes (Bucky)	75	7.5	75	ten	75	fifteen
Shoulder	Axillary (non grid)	65	3.7	65	4.5	65	6
Sinus	Caldwell	80	6	80	7.5	80	9
Sinus	Waters	80	7.5	80	9	80	10
Sinus	Lateral	80	3	80	3.7	80	4.8
Skull	AP	80	9	80	12	80	13.5
Skull	Lat (Grid)	80	3.7	80	4.8	80	5.5
Tib-Fib	AP (tt)	65	2.4	65	3	65	4
Tib-Fib	Lat (tt)	65	2	65	2.4	65	3
Toe	All Views	60	1.7	60	2.2	60	2.8
T-Spine	AP	80	6.5 - 9	80	15 - 18	80	28
T-Spine	Lat (2 sec)	85	11 to 16	85	25 - 28	85	45 - 50
Wrist	PA	60	1.3	60	1.5	60	1.7
Wrist	Obl	60	1.5	60	1.7	60	1.9
Wrist	Lat	65	1.5	65	1.7	65	1.9
Zygomatic Arch	SMV view (non grid-30")	65	7.5	65	9	65	11



Digital Radiography Solutions "Maximizing Image Quality, Minimizing Patient Dose"

Disclaimer: Digital Radiography Solutions (dRs) cannot be held responsible for any images taken or passed at your facility. Your radiologist or physician always has the final say if the image is of diagnostic value.