

KONICA CR TECHNIQUE CHART 100% More mAs

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Hip	AP (non grid)	70	6.4	77	8	70	10
Hip	X-Table Lat (Grid)	90	32 - 50	90	60 - 80	90	100 - 120
Humerus	AP	65	2.5	70	4	65	5
Knee	AP (Bucky)	75	5.4	85	7.2	75	8
Knee	Obl (Bucky)	75	4.8	85	6.4	75	7.2
Knee	Lat (Bucky)	75	4.8	85	6.4	75	7.2
Knee	Sunrise (non grid)	65	6	70	7.6	65	9.6
Knee	Non-Bucky	65	5.4	70	6	65	6.6
L-Spine	AP	80	14 - 22	90	28 - 36	80	44 - 52
L-Spine	X-Table Lat (Grid)	90	110 - 130	95	170 - 200	90	320 - 400
Mandible	Obl (40")	65	9	77	12	65	15
Pelvis	AP (Grid)	80	14.6	85	29	80	46
Ribs	Upper (72")	75	16 - 24	80	28 - 40	75	50 - 64
Ribs	Lower (40")	80	10 - 16	85	20 - 25	80	32 - 40
Ribs	Obl (72")	75	14 - 18	80	28 - 44	75	46 - 56
Shoulder	AP (non grid)	65	4.8	77	6	65	6.4
Shoulder	Y View - Bucky	75	15	85	20	75	30
Shoulder	Axillary (non grid)	65	7.4	77	9	65	12
Sinus	Caldwell	80	12	85	15	80	18
Sinus	Waters	80	15	85	18	80	20
Sinus	Lateral	80	6	85	7.4	80	9.6
Skull	AP	80	18	85	24	80	27
Skull	Lat (Grid)	80	7.4	85	9.6	80	11
Tib-Fib	AP	65	4.8	77	6	65	8
Tib-Fib	Lat	65	4	77	4.8	65	6
Toe	All Views	60	3.4	63	4.4	60	5.6
T-Spine	AP	80	13 - 18	90	30 - 36	80	56
T-Spine	Lat (2 sec)	85	22 - 32	90	50 - 60	85	90 - 100
Wrist	PA	60	2.6	66	3	60	3.4
Wrist	Obl	60	3	66	3.4	60	3.8
Wrist	Lat	65	3	70	3.4	65	3.8
Zygomatic Arch	SMV view (non grid- 30")	65	15	70	18	65	22



Digital Radiography Solutions "Maximizing Image Quality, Minimizing Patient Dose"

Disclaimer: Digital Radiography Solutions (dRs) cannot be held responsible for any images taken or passed at your facility. Your radiologist or physician always has the final say if the image is of diagnostic value.