

## KONICA CR OPTIMUM kV

Body Part	kV
Chest (Bucky/Grid)	<b>110-130</b>
Chest (Non Grid)	<b>80-90</b>
Abdomen	<b>75-80</b>
Abdomen (Iodine)	<b>70-75</b>
Extremities (Table Top)	<b>60-65</b>
Extremities (Grid)	<b>65-70</b>
Extremities (Bucky)	<b>70-75</b>
AP Spines	<b>75-80</b>
C-Spine Lateral	<b>75-90</b>
T-Spine Lateral	<b>75-85</b>
L-Spine Lateral	<b>75-90</b>
Ribs (Upper and Lower)	<b>70-80</b>
Skull	<b>75-80</b>
BE – (Regular)	<b>100-110</b>
BE – (Air Contrast)	<b>100</b>
<b>Pediatric:</b>	
Infant Extremities	<b>50-65</b>
Pediatric Chest (Non Grid)	<b>70-80</b>

Dose Exposure Saved							
SID	kV	mAs	Dose (mR)	Radiation Saved (%)	50% EI Decrease (mAs)	50% EI Dose (mR)	Total Dose Reduction (%)
40"	70	20	221.0				
40"	81	10	152.8	30.90%	5.0	76.4	65.43%
40"	85	8	134.8	39.00%	4.0	67.4	69.50%
40"	90	6.3	120.0	45.70%	3.2	60.0	72.85%
40"	96	4	87.0	60.60%	2.0	43.5	80.32%



Digital Radiography Solutions "Maximizing Image Quality, Minimizing Patient Dose"

Disclaimer: Digital Radiography Solutions (dRs) cannot be held responsible for any images taken or passed at your facility. Your radiologist or physician always has the final say if the image is of diagnostic value.