

KONICA CR TECHNIQUE CHART 66% More mAs

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Hip	AP (non grid)	70	5.5	70	7	70	8.8
Hip	X-Table Lat (Grid)	90	36 - 64	90	53 - 69	90	88 - 97
Humerus	AP	65	2.1	65	3.6	65	4.4
Knee	AP (Bucky)	75	4.8	75	6.4	75	6.9
Knee	Obl (Bucky)	75	4.4	75	5.7	75	6.4
Knee	Lat (Bucky)	75	4.4	75	5.7	75	6.4
Knee	Sunrise (non grid)	65	5.2	65	6.8	65	8.5
Knee	Non-Bucky	65	4.8	65	5.3	65	5.9
L-Spine	AP	80	12.7 - 20	80	25 - 32	80	39 - 47
L-Spine	X-Table Lat (Grid)	90	100 - 120	90	150 - 200	90	280 - 360
Mandible	Obl (40")	65	8	65	10.7	65	13.3
Pelvis	AP (Grid)	80	19.4	80	25.8	80	41
Ribs	Upper (72")	75	14.3 - 17.7	75	25 - 28.4	75	44 - 57
Ribs	Lower (40")	80	8.9 - 14.3	80	17.7 - 22.3	80	28 - 36
Ribs	Obl (72")	75	12.4 - 15	75	25 - 39	75	41 - 49
Shoulder	AP (non grid)	65	4.3	65	5.3	65	6.5
Shoulder	Y View - Bucky	75	13.3	75	17..33	75	27
Shoulder	Axillary (non grid)	65	6.5	65	8	65	10.7
Sinus	Caldwell	80	10.7	80	13.3	80	16
Sinus	Waters	80	13.3	80	16	80	17.7
Sinus	Lateral	80	5.3	80	6.6	80	8.5
Skull	AP	80	16	80	21.3	80	24
Skull	Lat (Grid)	80	6.5	80	8.5	80	9.7
Tib-Fib	AP	65	4.4	65	5.3	65	7.1
Tib-Fib	Lat	65	3.6	65	3.3	65	5.3
Toe	All Views	60	3.1	60	4	60	5.2
T-Spine	AP	80	6.5 - 8.1	80	27 - 32	80	49
T-Spine	Lat (2 sec)	85	20 - 28	85	44 - 49	85	80 - 90
Wrist	PA	60	2.3	60	2.7	60	3.1
Wrist	Obl	60	2.7	60	3.1	60	3.3
Wrist	Lat	65	2.7	65	3.1	65	3.3
Zygomatic Arch	SMV view (non grid - 30")	65	13.3	65	16	65	20



Digital Radiography Solutions "Specializing in Educational Conferences and Seminars"

Disclaimer - Digital Radiography Solutions (DRS) cannot be held responsible for any images taken or passed in your facility. Your radiologist always has the final say if an image is diagnostic.