

CESIUM DR UNIVERSAL TECHNIQUE CHART

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Abdomen	AP (Grid)	85	4 to 5	85	8 to 10	85	16 -20
Ankle	AP	70	1.5	70	2	70	2.5
Ankle	Obl	70	1.3	70	1.8	70	2
Ankle	Lat	70	1	70	1.25	70	1.5
Chest -Adult	AP (Grid)	117	1.6	117	2	117	3.2
Chest -Adult	AP (Non Grid)	90	1	90	1.6	90	2
Chest (2-9 lb)	AP (Non Grid - 45")	71	1.0	73	1.3	75	1.5
Chest (2-9 lb)	Lat (Non Grid - 45")	73	1.4	75	1.6	77	2
Chest (10-20 lb)	PA (Non Grid - 72")	80	0.5	80	0.6	80	0.8
Chest (10-20 lb)	Lat (Non Grid - 72")	80	1.0	80	1.3	80	1.6
Chest (25-35 lb)	PA (Non Grid - 72")	82	0.8	82	1.0	82	1
Chest (25-35 lb)	Lat (Non Grid - 72")	82	1.6	82	2.0	82	2
C-Spine	AP (Bucky - 72")	85	6.0-8.	85	8.0-11	85	11.0-14
C-Spine	AP (Bucky - 40")	85	2	85	3	85	4
C-Spine	Odontoid (72")	85	8.0	85	11.0	85	14
C-Spine	Odontoid (40")	85	2.5	85	3.5	85	4.5
C-Spine	Lat (Bucky - 72")	85	4	85	6.3	85	8
C-Spine	Swimmers (40")	90	12.5	95	16.0	95	25
Elbow	AP	66	1.0	66	1.2	66	1.4
Elbow	Obl	66	1.2	66	1.4	66	1.6
Elbow	Lat	66	1.4	66	1.6	66	1.8
Finger	All Views	60	0.63	60	0.7	60	0.8
Foot	AP	66	1.0	66	1.2	66	1.6
Foot	Obl	66	1.2	66	1.4	66	1.8
Foot	Lat	66	1.8	66	2.2	66	2.8
Forearm	AP	68	1.2	68	1.6	68	2.2
Forearm	Lat	68	1.4	68	1.8	68	2.5
Hand	PA	63	0.6	63	0.8	63	1
Hand	Obl	63	0.8	63	1	63	1.2
Hand	Lat	66	1	66	1.2	66	1.6



Digital Radiography Solutions "Specializing in Educational Conferences and Seminars"

Disclaimer - Digital Radiography Solutions (DRS) cannot be held responsible for any images taken or passed in your facility. Your radiologist always has the final say if an image is diagnostic.