

# GADOLINIUM DR UNIVERSAL TECHNIQUE CHART

Part	View	Small		Medium		Large	
		kV	mAs	kV	mAs	kV	mAs
Hip	AP	85	5 to 7	85	10 to 12	85	14 to 16
Hip	X-Table Lat (Grid)	90	20	90	32	90	50
Humerus	AP (Non Grid)	66	2.2	66	2.8	66	3.2
Knee	AP (Bucky)	77	4	77	6.4	77	8
Knee	Lat (Bucky)	77	3.2	77	5	77	6.4
Knee	Sunrise	70	2.4	70	3	70	3.8
Knee	Non-Bucky	70	3.8	70	4.5	70	5.4
L-Spine	AP	90	7	90	12	90	20
L-Spine	X-Table Lat (Grid)	95	25	95	36	95	60
Mandible	Obl (40")	81	10	81	12	81	15
Pelvis	AP (Grid)	85	2.5	85	16	85	20
Ribs	Upper AP (72")	81	8	81	6	81	25
Ribs	Upper Obl (72")	81	12	81	25	81	35
Ribs	Lower AP (45")	85	8.0	85	16	8	25
Shoulder	AP (Bucky)	77	6.3	77	10	77	14
Shoulder	Mercedes	77	12	77	20	77	32
Shoulder	X-T Axillary (N-Grid)	70	4.5	70	5.5	70	6.3
Sinus	Caldwell	85	6	85	8	85	10
Sinus	Waters	85	7	85	9	85	12
Sinus	Lateral	85	3	85	4	85	5
Skull	AP	85	6	85	8	85	10
Skull	Lat (Grid)	85	3	85	4	85	5
Tib-Fib	AP	77	3.2	77	4	77	5
Tib-Fib	Lat	77	2.5	77	3.2	77	4
Toe	All Views	60	0.8	60	1	60	1.2
T-Spine	AP	85	7	85	11	85	Sixteen
T-Spine	Lat	90	16	90	25	90	36
Wrist	PA	63	1.2	63	1.5	63	1.8
Wrist	Obl	63	1.4	63	1.8	63	2.2
Wrist	Lat	67	1.8	67	2.2	67	2.8
Zygomatic Arch	SMV view	70	8-Jan	70	10	70	12



Digital Radiography Solutions "Specializing in Educational Conferences and Seminars"

**Disclaimer** - Digital Radiography Solutions (DRS) cannot be held responsible for any images taken or passed in your facility. Your radiologist always has the final say if an image is diagnostic.